



CREATE IT.

**iCare**

AUTOMATICALLY CLEANS & LUBRICATES

**NAKANISHI INC.**   
[www.nsk-inc.com](http://www.nsk-inc.com)

700 Shimohinata Kanuma-shi Tochigi 322-8666, Japan  
TEL : +81 (0)289-64-3380 FAX : +81 (0)289-62-5636

**NSK Oceania Pty Ltd**  
[www.nskoceania.com.au](http://www.nskoceania.com.au)

Unit 22, 198-222 Young St, Waterloo, Sydney, NSW 2017, Australia  
TEL : +61 (0)2 8306 3000 FAX : +61 (0)2 8306 3030

Specifications are subject to change without notice.

 PR-D901E Ver.1 14.03.00.00

# iCare

iCare sets a new standard for handpiece maintenance systems. In one stylish and compact body, it offers you both ease of use and multiple functions to achieve complete cleaning and lubrication. The proper solution volume is preset based on handpiece type, and maintenance can be initiated at the touch of a button. The chuck cleaning system keeps chucks pristine, and the purging select mode lets you alter purge time depending on the situation for truly comprehensive maintenance. Thanks to its stylish and compact body, iCare is also space efficient and totally in harmony with your working environment.

## ADVANTAGES

- Preset solution volume based on handpiece type (high-speed handpieces, low-speed handpieces)
- Thorough cleaning and lubrication with simple operation
- Chuck cleaning function ensures that the chuck is thoroughly cleaned and lubricated
- Purging select mode ensures complete removal of excess oil
- Integrated large oil tank (1.2 litre) reduces burden of refill



# AUTOMATICALLY CLEANS & LUBRICATES

## PRESET PROPER SOLUTION VOLUME BASED ON HANDPIECE TYPE

The proper solution volume is preset based on high speed and low speed handpieces, ensuring precise and thorough lubrication at the touch of a button.




**ERGONOMIC AND EASILY READABLE LARGE PANEL**  
Simple One-Touch Operation to Start Lubrication.




**PURGING SELECT MODE**  
After lubrication, you can select from appropriate modes offering different purge lengths.

## LOW SPEED HANDPIECES

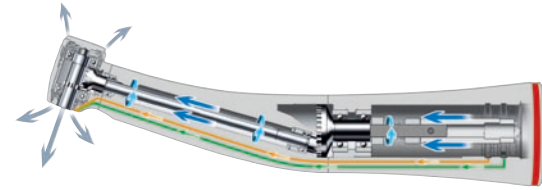


Low-speed handpieces incorporate gearing systems that require extra solution penetration. The NSK rotation system guarantees optimum oil delivery to the internal components of all low-speed handpieces.

## HIGH SPEED HANDPIECES

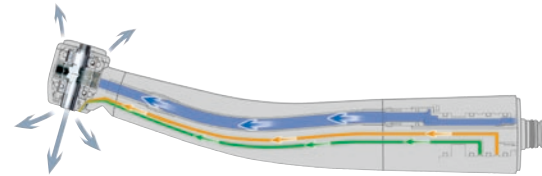

iCare meticulously cleans the components of high-speed handpieces.

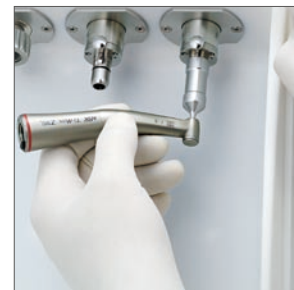

## CHUCK CLEANING FUNCTION

The chuck cleaning function completely removes dirt and cutting powder from the chuck.



## EASY TO CLEAN

iCare's front door can be completely removed to allow fast, easy cleaning of the system.



## SOLUTION TANK

iCare's big solution tank has a capacity of 1.2 liters, allowing you to pour in a full one-liter bottle of maintenance solution at once, reducing the necessity for frequent refilling. The enclosed nozzle enables clean and easy refills.



## HP HEAD FILTER PREVENTS MIST LEAKS (Optional)

HP Head Filter is made to absorb any spray mist generated, which prevents mist leaks.

## ACCOMMODATES BOTH NSK HANDPIECES AND THOSE OF OTHER BRANDS

NSK and all other brands of low speed handpieces connect directly to the coupling supplied. NSK high speed handpieces require the PTL adaptor. Adaptors are available for direct connection of most other brands of high speed handpieces.



## SAFE OPERATION WITH A RANGE OF ALARM FUNCTIONS

The alarm LED ensures safe and secure maintenance by alerting you to various situations, including inadequate solution quantity and an unlocked front door.

# AUTOMATICALLY CLEANS & LUBRICATES




## STYLISH AND COMPACT

In the pursuing ultimate ease of use, NSK has given iCare the power to perform automatic maintenance for up to four handpieces at once. iCare is also designed to be simple to set up and space efficient, you can install it virtually anywhere.



## iCare COMPLETE SETS


### Dual Standard Ports + Dual Rotation Ports

MODEL	ORDER CODE	
iCare C2 Type	Y1002790	

### CONTENTS

- iCare unit
- Power cord
- Maintenance oil for iCare (1 Litre)
- Mist filter set
- Oil absorber sheet
- Air tube
- Chuck Cleaning Nozzle

### Single Standard port + Triple Rotation ports

MODEL	ORDER CODE	
iCare C3 Type	Y1002791	

### SPECIFICATIONS

- Power Supply : AC 100-240V 50/60Hz
- Air Pressure : 0.5 - 0.6 MPa (5.0 - 6.0 kgf/cm<sup>2</sup>)
- Weight : 6.5 kg (C2 type/C3 type)
- Dimension (mm) : W280 x D240 x H350
- Solution Tank Capacity : 1,200 ml
- \* Connects only to Midwest 4-hole type
- \* Air piping is necessary (5.0 - 6.0 kgf/cm<sup>2</sup>)

## iCare ADAPTORS

MODEL	ORDER CODE	
PTL Adaptor	Z257010	• For NSK High Speed Handpieces for PTL-CL-LED Coupling
B2/M4 Adaptor	T904	• For Borden 2 hole High Speed Handpieces
KV Adaptor	Z257020	• For KaVo® High Speed Handpieces for MULTiflex® LUX Coupling
SR Adaptor	Z257023	• For Sirona® High Speed Handpieces for Quick Coupling
WH Adaptor	Z257021	• For W&H® High Speed Handpieces for Roto Quick Coupling
BA Adaptor	Z280022	• For Bien-Air® High Speed Handpieces for Unifix® Coupling
F-Type Adaptor	Z280042	• For ENDO-MATE TC2, DT / TASKAL Wizard Heads

Check with the dealer for the adaptors to mount other manufactures' handpieces.

## SPARE PARTS

MODEL	ORDER CODE	
Maintenance Oil for iCare 1 Bottle, 1 Litre	Z016130	
Maintenance Oil for iCare Pack of 6 Bottles	Y1002783	
Mist Filter Set Pack of 12	Y900159	
Oil Absorber Sheet Pack of 10	U433135	
Air Filter	U508352	
Chuck Cleaning Nozzle	Z257301	
HP Head Filter Pack of 200	U1120118	

KaVo® and MULTiflex® LUX are registered trademarks of Kaltenbach & Voigt GmbH & Co. Sirona® is a registered trademark of Sirona Dental Systems GmbH. W&H® is a registered trademark of W&H Dentalwerk Bürmoos GmbH. Bien-Air® and Unifix® are registered trademarks of Bien-Air Dental S.A.