OPERATION MANUAL

Tip Power Guide

NAKANISHI INC. www.nsk-dental.com
700 Shimohinata, Kanuma, Tochigi 322-8666, Japan

NSK Europe GmbH
Elly-Beinhorn-Strasse 8, 65760 Eschborn, Germany

Visit our website

Issued Date: 2021.04.29
Printed Date: 2021.04.30 0217

Specifications are subject to change without notice.

EN This operation manual describes the procedure of mounting/removing and reprocessing the tip. Carefully read the operation manual of the Ultrasonic System "VarioSurg" and "VarioSurg 3" for other procedures.

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EN • Only hold the tip at the base where it screws on to the handpiece. Otherwise there is a risk of injury to your hand or tip damage. • When attaching the tip, always wear sterilized gloves, and use a tip, handpiece, and tip replacement wrench that are all sterilized.

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EN • Refer to the operation manual for the Ultrasonic System "VarioSurg" and "VarioSurg 3" that the handpiece and tip are to be connected, to set the power level. • Refer to the table 1 of this manual which defines the maximum power level of the tips according to the Ultrasonic System "VarioSurg" and "VarioSurg 3" which is being used.

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EN • During its operation, the whole tip vibrates due to ultrasonic waves. Do not bring any part of the tip into contact with soft tissue, gums or skin. Regardless of the use of coolant water the tip in vibration could cause a burn injury.

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EN • If the tip doesn't vibrate when the device is being activated, remove the handpiece from the patient's mouth, then restart the handpiece operation. If this does not improve the situation, the tip may have been loosened during use. Check that the tip is securely attached using the correct tool. • Use the tip ONLY in the correct mode and never exceed the maximum power defined for the tip by this manual. If used outside the prescribed power range, the tip may break, and the tooth surface or soft tissue may be damaged.

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EN • Refer to the operation manual for the Ultrasonic System "VarioSurg" and "VarioSurg 3" that the handpiece and tip are to be connected, to set the power level.

1) Set the power of the Ultrasonic System "VarioSurg" and "VarioSurg 3" at the lower end of the permitted range and increase gradually depending on the patient condition.

2) Adjust the irrigation flow in order for the coolant water to reach the tip end.

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
Model			Mode		
SG1				80%	B
SG1A				80%	B
H-SG1				150%	B
SG2R				80%	
SG2L				80%	
SG3				80%	
SG4				80%	
SG5				80%	
SG6D			✓	80%	
SG7D			✓	50%	
SG8				80%	B
SG8A				80%	B
H-SG8				150%	B
H-SG8R				150%	A
H-SG8L				150%	A
SG9				50%	
SG10				50%	
SG11				50%	
SG14R				80%	
SG14L				80%	
SG17				80%	
SG18R				50%	
SG18L				50%	
SG19				80%	B
SG30				50%	
SG58				80%	B
SG68				80%	A
SG15A			✓	50%	C
SG15B			✓	50%	C
SG15C			✓	50%	D
SG15D			✓	50%	D
SG16A			✓	50%	C
SG16B			✓	50%	C
SCL1*				50%	E
SCL1D*				80%	E
SCL2D*			✓	80%	E
SCL3*			✓	50%	E
SCL3D*			✓	80%	E
SCL4D*			✓	80%	E
SCL5*			✓	50%	E
SCL5D*			✓	80%	E
E30RD-S			✓	50%	
E30LD-S			✓	50%	
E31D-S			✓	50%	
E32D-S			✓	50%	
G1-S				50%	
P20-S				100%	
P25R-S				100%	
P25L-S				100%	

EN *Refer to OM-DZ0916MA for how to use the Tips.
DE *Informationen zur Verwendung der Aufsätze finden Sie unter OM-DZ0916MA.
FR *Référez-vous au document OM-DZ0916MA pour connaître le mode d'utilisation des inserts.
ES *Consulte OM-DZ0916MA para obtener información sobre cómo usar las puntas.
IT *Consultare OM-DZ0916MA per informazioni sull'utilizzo delle punte.
PT *Consulte OM-DZ0916MA sobre como utilizar as pontas.


REF 10000977


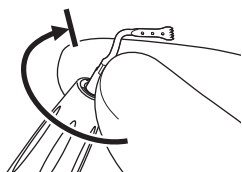
1

2

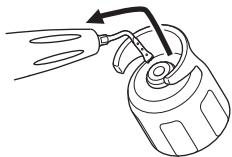
3

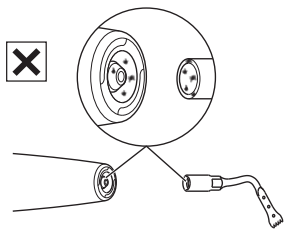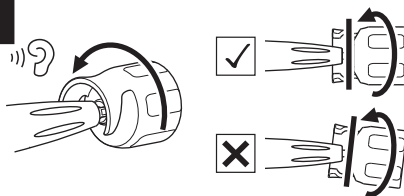REF Z305350

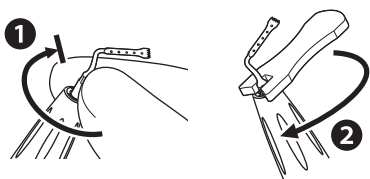1

EN SG68 is effective for the following applications.

- Cases where cutting is performed in a vertical direction to the alveolar crest line
- Cases where access is difficult using a regular tip shape, such as for the general molar area or the outside lower edge area of the mandibular ramus

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- Cases of restricted areas or patients with small mouth openings

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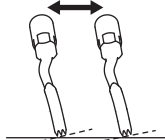
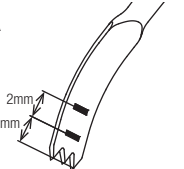
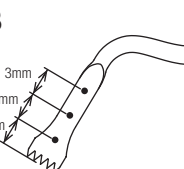
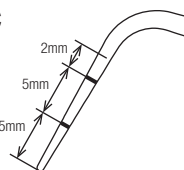
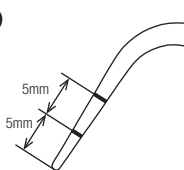
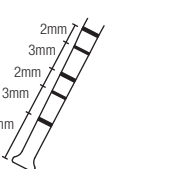
EN • Before using an alcoholic product for internal disinfection, the Tip must first be cleaned with a non-protein-fixing product (e.g. ALPRO WL-clean). • Clean the tip and the tip wrench prior to disinfection. If blood remains on the internal surface it can become clotted and cause product failure. • Use only cleaning/disinfectant products listed in this manual. • Follow any additional local directives/standards and guidelines for cleaning, disinfection, and sterilization. • Do not clean, immerse, or wipe with electrolyzed-oxidizing water (strong acidic water, super acidic water), strong acidic and strong alkaline liquid medicine, solvent containing chlorine, benzene, or thinner. • Use clean and dry compressed air for cleaning. • Tips must be removed from the washer-disinfector immediately (within 1 hour) after the cleaning, disinfecting and drying cycle is complete to prevent corrosion. • After using washer-disinfector, dry the product completely. The remaining moisture causes internal corrosion etc. • Do not autoclave the tip and the tip wrench with other instruments even when it is in a pouch. This is to prevent possible discoloration and damage to the product from chemical residue on other instruments. • Do not heat or cool the product too quickly. Rapid change in temperature could cause damage to the product. • Immediately after sterilization is complete (within 1 hour), remove the tip and the tip wrench from the sterilizer. Failure to do so may cause corrosion.

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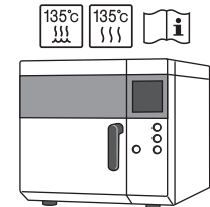
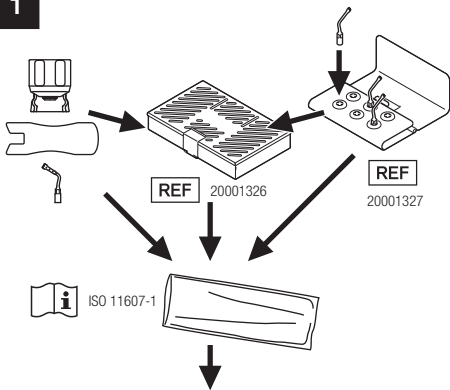
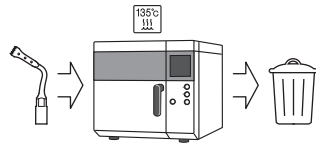
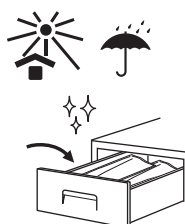
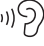
EN • This tip is specially designed for the Ultrasonic System "VarioSurg" and "VarioSurg 3" by NSK. Do not use with other systems. • Be sure to always carefully read the operation manual for the Ultrasonic System before use to understand the usage and performance of the instrument. • Be sure to always use with sufficient coolant water. Insufficient coolant water could cause a burn of the affected area or the overheating of the instrument. • Surface temperature of tip shall be more than 65 degree without using coolant water. To avoid this event, be sure to use coolant water. • Be sure to always use the tip at a setting not exceeding the maximum power level. If the tip is used in excess of the maximum power, the tip could break or excess cutting could occur. • Do not touch the top part of the tip to avoid an injury.

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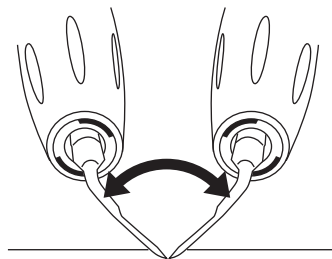
fig. A

EN • This product must only be used by a qualified personnel in accordance with the indications for use and the directions for use stated in the operation manual of the Ultrasonic System "VarioSurg" and "VarioSurg 3". It is attached to and this operation manual. Do not use it for any other purpose. • Operators and all others in the area must wear eye protection and a mask when operating this tip. • Prior to clinical use, inspect the tip outside the patient's oral cavity. Check for vibration, noise and overheating. If any abnormalities are found, stop using the tip immediately and contact your Authorized NSK Dealer. • The tip is a consumable item and wears with use. If it is worn, abnormal noise, weak or no vibration, or damage may occur. Replace the tip with a new one when it is worn, or loses diamond on the tip surface (only for diamond coated tips). Also, the tip can break due to metal fatigue. Replace with a new tip on a regular basis. • The coating on the tip surface (TiN coating) may peel off depending on the number of applications (typically around 5 applications) and the bone condition. If the coating peels off, the smooth glide of the tip surface becomes less effective, resulting in reduced cutting performance. If you sense this type of condition, replace with a new tip. • The tip may be discolored by repeated reprocessing. If the tip is discolored significantly, replace it with a new one. For the diamond coated tip, diamond on the tip surface may partially disappear and discoloration occurs. In this case, replace the tip with a new one. • Do not grind or bend the tip. It may break or may not vibrate during use. • Keep away from patients with cardiac pacemakers. • The tip is delivered in a non-sterile condition and must be autoclaved prior to use. • For irrigation, avoid use of ozonated water, oxidizing water (strongly acidic water, superacidic water, electrolyzed water, etc.), or hypochlorous solution. Doing so may lead to clogging, corrosion of metal parts, etc. • Even if the power level is correctly set for the Tip in use, application of excessive pressure to the Tip may damage it. Should Tip breakage occur, immediately evacuate the oral cavity of any Tip pieces using strong suction. • During vibration, do not hold the tip nor make contact with the tip for an extended time. • If there are abnormalities in water supply during operation, stop using the tip immediately and replace it with a new one. • If the tip cracks or breaks during use, immediately stop using the tip, and replace with a new tip. Operation with a damaged tip could injure the gums, periodontal tissue, or other locations. • Do not apply more pressure than required to the tip end. This could cause the tip to break or bend. • Refer to the operation manual of the Ultrasonic System for EMC information. • Report all serious events related to the product immediately to the manufacturer and responsible national authority. • Be aware of the tip end position. Constantly move the tip so that it does not stagnate at one point. If the tip is stagnated at a single point, there will not be enough cooling, and this could result in overheating of the bone or excessive cutting of the bone. • Do not bring the tip into contact with soft tissue such as gums or mucous membranes for an extended period of time. This could result in a burn injury. • Start with a low power setting within the power range, observing the patient's reaction and according to the treatment plan. • Before use, check that the tip is firmly attached to the handpiece. • Do not twist or strain the tip, and do not use it as a replacement for surgical tools. This could cause the tip to deform or break. • During the cutting, DO NOT push down the Tip left and right (shown on the fig. A). There is a possibility the point of the tip bending. (SG30)

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